



(€

Convert your VT1® Master or Master GT to an automated device. VisioClick also adapts to previous versions (from 2012).

Compact, lightweight, and operational from the first connection. A simple update of the VT1® software is all it takes.

No operator presence required during tests. Multilingual voice instruction ensures global understanding.

HOW

DOES IT WORK?

4-step tests. No operator intervention

STEP 1: HEADPHONES

The patient puts on the headphones and receives test instructions displayed in the VT1®. Multilingual Voice Instructions available.

STEP 2: RESPONSE UNIT

Patient responds directly to instructions by clicking on his unit.

STEP 3: RECORDING

Results are automatically saved.

STEP 4: END OF TEST
Information is given to the patient and operator indicating the test is completed. Results are recorded and can be interfaced with all «business software».



Increases lifespan of earpadsNo sound reductionA more hygienic solution

NEW!

EARPAD COVERS PACK

VisioClick® delivered with a pack of earpad covers.

Single-use biocompatible for all your tests.

Benefits:



- Windows XP SP3, Windows Vista, Windows 7, Windows 8 and Windows 10
- Interfacing with market software packages (OHM, Cority...)

Specific software features:

- Device can perform all tests individually in the order of your choice or a preprogrammed entire test protocol
- Creation of customizable profession-related sequences that can be performed in automatic mode
- Multilingual voice instructions (French, English, German, Spanish, Dutch, Italian, Polish, Portuguese, Turkish, Arabic, etc.)

YOUR DISTRIBUTOR

WEIGHT CONNECTIVES

WARRANTY 2 years (without headphones support) 9x5x2 inch

1.55 lbs

Jack 0.12 inch (to audio headphones), USB Hub (to VT1® device)

POWER SUPPLY VOLTAGE 5VDC (via USB)

REFERENCE STANDARDS EN60601-1, EN60601-1-2
MARKING CE

MEDICAL CLASS Classe I (directive 93/42/CEE)



550 Pharr Road, Suite 201 Atlanta GA 30305

Phone: +1 800 496 8290 Fax: + 1 800 496 8290 contact@depisteo.com

 $C \in$